

# myActive Alert

Independence Solution



## What is it?

The Tunstall myActive Alert features patented and clinically validated automatic fall detection algorithms. Additional key features and benefits of the technology include patented algorithms designed to reduce false alarms, monitoring the number of steps taken, and notifying providers when the pendant is not worn by the subscriber. The myActive Alert also incorporates a water resistant HELP button to request help at the press of a button.

## How does it work?

By pressing the button on the pendant, a help call is generated through the console unit and sent to the 24-hour Emergency Response Center (ERC). Trained operators can access call details (subscriber's name, address, and relevant information) that are displayed on their screen.

Our ERC operator communicates with the subscriber through the console's loudspeaker and microphone to determine the nature of the call and the appropriate course of action.

Built-in automatic fall detection senses when the subscriber has fallen and triggers a call to the ERC.



## Who is it for?

A wide variety of subscribers can benefit from the myActive personal emergency response pendant, including people who:

- Are at high risk for falls
- Have muscle weakness, especially in their legs
- Take medications that may cause dizziness
- Have hypotension (low blood pressure)
- Have neurological conditions such as Parkinson's Disease or Epilepsy
- Have mental, physical, or sensory impairments

The myActive offers family, caregivers, and friends the reassurance that their loved ones can use their myActive pendant to easily call for help whenever they need it from anywhere in and around the home.

# myActive

## Key Features

- Advanced algorithms
- Adjustable fall sensitivity
- Convenient cancel button
- Audible announcements available in English or Spanish
- Small, lightweight design
- Ergonomic alert button
- Compatibility
- 390ft (120m) radio range in free space
- Water resistant (IP67)
- Multiple wearing options
- Supervised auto presence
- FDA registered and ETL listed

## Benefits

- Enables the myActive to distinguish between fall and non-fall events
- Customizable to suit individuals
- Enables the wearer to cancel fall activations if assistance is not required, eliminating unnecessary calls
- Subscriber is informed of the call progress and status
- Discreet and easy to carry
- Enables subscribers with visual impairments or limited dexterity to trigger a call
- The myActive can be easily programmed to the Tunstall CEL MXD
- Range suitable for typical dwellings/homes
- Can be worn in shower or bath
- Necklace or on a belt
- Notifies the Emergency Response Center if the myActive is not seen/ within range of the console for a specified period of time
- Tunstall is FDA registered, complies with all medical device regulations, and is ETL listed to conform with UL 1635 and UL 1637 product safety standards

## Specifications

• Part Number:	MT002
• Dimension:	2.165" x 1.468" x 0.603"
• Weight:	1oz
• RF Transmission Frequency:	418 MHz
• Operating Temperature:	-10°C to 60°C
• Battery Type:	CR2450 Lithium
• Battery Life:	1 year or more (2 years in Basic non-supervised mode).
• Battery Voltage:	3.0V
• FCC ID:	NY5MAA01
• IC:	9663A-MAA01
• Product Compatibility	CEL MXD

For additional information please call

**1.800.728.3775**

or [info@saversalert.com](mailto:info@saversalert.com)

[www.saversalert.com](http://www.saversalert.com)

Due to the many types of falls that can occur, the user should always press the HELP button whenever help is needed and not rely solely upon the determination of the fall detection algorithms.\*

MA-17-0005 Rev 1.0

